promed PBM-4.2

(Model: U80EH)

Oberarmblutdruckmessgerät
Upper Arm Blood Pressure Meter
Tensiomètre de bras

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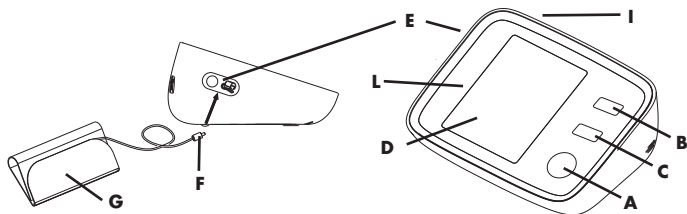
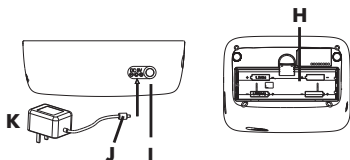
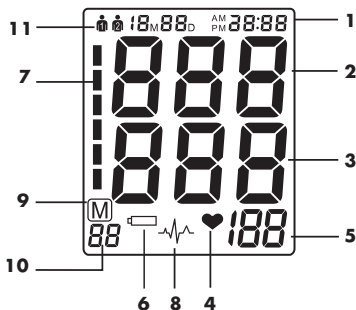
SCOPE OF DELIVERY/ACCESSORIES

The following components are included in the scope of delivery of your **Promed PBM-4.2** blood pressure monitor:

- 1x blood pressure monitor
- 1x arm cuff (arm circumference 22 - 42 cm)
- 1x tube connecting piece
- 4x batteries, 1.5 V AA

The following accessories and spare parts are available at Promed:

1. Accessories:

- Power adapter item no.: 402093

2. Spare parts

- Arm cuff (arm circumference 22 - 42 cm) item no.: 402061
- Batteries, 1.5 V AA item no.: 359993

DESCRIPTION OF THE UNIT

Please keep this user manual in a safe place!

Please read the user manual thoroughly before using the device for the first time.

Many thanks for purchasing the **Promed PBM-4.2** blood pressure meter. You have acquired a high-quality medical product for the personal checking of health.

Promed is a leading company with decades of experience in personal hygiene, wellness and health areas.

The **Promed PBM-4.2** unit has been designed and manufactured in accordance with the Medical Device Directives 93/42/EEC to guarantee quality during use. The device may be used once this user manual has been read.

We, as the manufacturer, cannot be made liable in any way for injury or damages to people or objects that arise from failure to comply with this user manual. We wish you a lot of enjoyment with your new **Promed PBM-4.2**. In the following we would like to familiarise you with the **Promed PBM-4.2** blood pressure meter.

Please read the user manual thoroughly before using the device for the first time.

You can receive professional advice wherever **Promed** products are sold or get in touch with us if you have any questions. We can provide you with the name of the representative responsible for you.

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Lindenweg 11, D-82490 Farchant, Ph: +49 (0) 8821/9621-0,

Fax: +49 (0) 8821/9621-21, E-Mail: info@promed.de

Further informations can be found on our homepage:

www.promed.de

DE

EN

FR

IT

ES

NL

RU

PL

SE

AR

INTRODUCTION

The **Promed PBM-4.2** blood pressure meter uses the oscillometric method of blood pressure measurement and is thus particularly suitable for the personal measurement and monitoring of blood pressure. With the oscillometric measurement method the meter measures the flow of your blood through the upper arm artery and converts these into readable measured values. In addition to this, it is not necessary to use a stethoscope with this measurement technique. The **Promed PBM-4.2** is an automatic blood pressure meter that can measure the systolic, diastolic pressure and pulse. This device is only intended for use by adults. An intelligent control system for the pumping process ensures that the unpleasant feeling caused by incorrect pumping is reduced and shortens the measuring time. In this way, the useful life of the arm cuff is substantially extended. Each measurement result is displayed on the screen and automatically saved. 2x90 memory spaces are available for this. The **Promed PBM-4.2** possesses a WHO (World Health Organisation) blood pressure classification index with which you can interpret the measured values more easily. Before using the **Promed PBM-4.2** blood pressure meter, please read through the operating instructions carefully and keep the operating instructions in a safe place.

What is irregular heartbeat?

The **Promed PBM-4.2** blood pressure meter can measure blood pressure and pulse even in the event of an irregular heartbeat. An irregular heartbeat is defined as a heartbeat that differs by at least 25 % from the average of all the heartbeats during the blood pressure measurement. It is important that you sit still and relaxed during the measurement and do not speak.

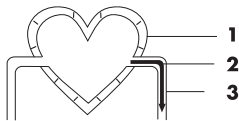
Please note: If, when measuring, you frequently see the symbol for irregular heartbeat  we recommend that you make sure that you see a doctor.

What is blood pressure?


Blood pressure is the force exerted by the blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands. Blood pressure is measured in millimetres of mercury (mmHg). The natural blood pressure of a person is represented by blood pressure at rest. This is obtained by measuring blood pressure in the morning before getting up.

Systolic blood pressure

1. Tensing of the heart muscle
↓
2. The blood is "pressed" out of the heart
↓
3. The blood pressure in the vessels rises
= high pressure

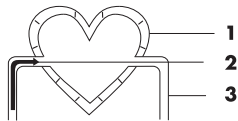Diastolic blood pressure

1. Relaxing of the heart muscle
↓
2. The blood runs into the heart
↓
3. The blood pressure in the vessels decreases
= low pressure

INTRODUCTION

What is high blood pressure and how can this be controlled?

High blood pressure is abnormally high arterial blood pressure which, if it remains uncontrolled, can cause many health problems such as stroke and heart attack. High blood pressure can be regulated by means of lifestyle, the avoidance of stress and medication under medical supervision. In order to prevent high blood pressure or keep it under control, the following principles should be adhered to:

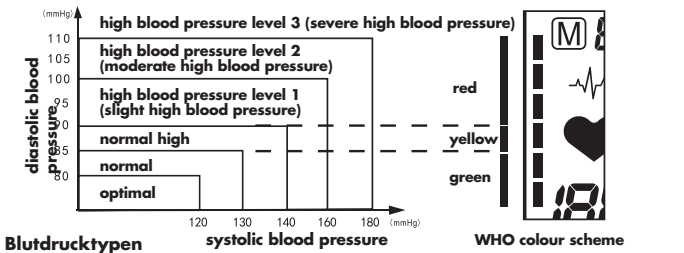
- do not smoke
- regularly do a sport
- eat less salt and fat
- go to the doctor regularly
- keep to the correct weight

Why should one measure one's blood pressure at home?

Visiting a clinic or doctor's practice can lead to nervousness and thus produce increased blood pressure. The measured values can be 25 to 30 mmHg higher than in a measurement at home. Measuring at home reduces the effects of other influences on the blood pressure measurement and thus supplements the measurements taken at the doctors and show the course of blood pressure more accurately and completely.

The blood pressure classification index of the WHO

Standards for assessing high blood pressure, without taking age into account, have been specified by the World Health Organisation (WHO) in accordance with the following table.

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
systolic blood pressure

WHO colour scheme

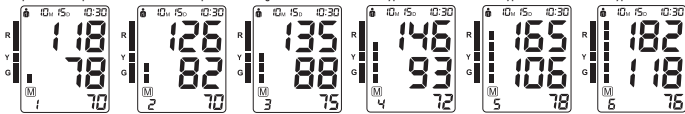1. Optimal blood pressure 2. Normal blood pressure 3. High normal value 4. Mild hypertension 5. Moderate hypertension 6. Severe hypertension

OPERATING THE UNIT

Testing/replacement of the battery

Remove the cover of the battery compartment (H) and insert the batteries. Ensure that the batteries are inserted with right polarity (+/-). If the batteries are too weak, this will be shown by the device on the battery status display  (6). As soon as this symbol lights up the batteries absolutely must be replaced, because otherwise the measurement result will be incorrect.

Caution:

1. Remove the batteries if the device is not in use for long periods of time.
2. Do not mix old and new batteries or different types of batteries.
3. Warning: If batteries leak and come into contact with the skin or eyes, wash immediately with copious amounts of water.
4. Batteries must be handled by an adult. Keep batteries out of the reach of children.
5. Only batteries of the same or equivalent type are recommended.
6. Remove exhausted batteries from the unit.
7. Dispose of batteries safely according to battery manufacturer's instructions.
8. Do not use the batteries after the expiry date.

Using a mains adapter

The device can also be operated with a mains adapter. Make sure that the adapter is permitted for use with medical products and meets the following technical requirements; output voltage: 6 VDC, output current: 600 mA, mains barrel connector: plus internal, external diameter 5.5 mm internal diameter 2.1 mm. You can always purchase a tested mains adapter from Promed.

Please note:

The batteries are not used when using a power adapter.

Please note:

If, during a measurement, the electricity supply is interrupted or the battery status display **(6)** lights up, then the measurement absolutely must be repeated.

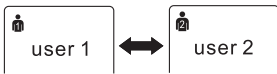
Connection of the upper arm cuff

In order to connect the upper arm cuff **(G)** to the device, insert the tube connecting piece **(F)** into the tube connection **(E)**. While doing this, do not use excessive force to insert the connecting piece in the device.

Setting mode

1. User setting mode

Press the "Set" setting button on the screen, and "User 1" or "User 2" is displayed. You can choose between "User 1" and "User 2" by pressing the "MEM" save button **(C)**. Your setting is saved by pressing the "SET" setting button **(B)** again.

2. Setting the date and time

After setting the user, you can now set the date. When 20XX flashes on the display, you can set the year by pressing the save button "MEM" **(C)**. With each press of the button, the year increases by 1; values from 2020 to 2099 are possible. You can save your setting by pressing the "SET" button **(B)** once more.

DE

EN

FR

IT

ES

NL

RU

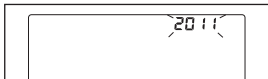
PL

SE


AR

OPERATING THE UNIT


Next, you can set the month. xxMxxD and xx:xx now appears on the display and xxM flashes. You can now set the month by pressing the save button "MEM" (C). With every press of the button the value increases by 1; values from 1 to 12 are possible. You can save your setting by pressing the "SET" button (B) once more.

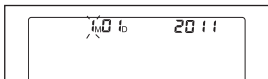
xxD now flashes on the display. To set the tag, press the save button "MEM" (C). With each press of the button, the value increases by 1; values from 00 to 31 are possible. You can save your setting by pressing the "SET" button (B) once more.

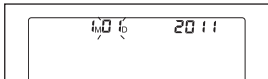
Next, you set the time. xxMxxD and xx:xx now appear on the display and xx: flashes; first, you can set the hour by pressing the save button "MEM" (C). With every press of the button, the value increases by 1; values from 1 to 23 are possible. You can save your setting by pressing the "SET" button (B) once more.

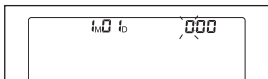Now flashes on the display: xx. To set the minutes press the save button "MEM" (C). With each press of the button, the value increases by 1; values from 00 to 59 are possible. You can save your setting by pressing the "SET" button (B) once more. "CL" now appears, your setting is completed by pressing the "SET" setting button (B) again.

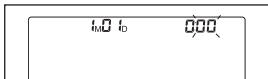3. It is possible to change users before each measurement. To this end, press the "Set" setting button (B) 2x and select the user by pressing the "MEM" save button (C). To leave setting mode and begin the measurement, you now press the START/STOP button (A).

Before the measurement

Before you carry out a blood pressure measurement you should remain entirely calm for 5 - 10 minutes, not eat, not drink any alcohol, not smoke, not do any sport and not bathe. All of these factors influence the measurement result. Remove items of clothing that are tight on your upper arm. Always measure on the same arm (normally left). Carry out the measurement regularly at the same time of day, because blood pressure changes during the day.

DE

EN

FR

IT

ES

NL

RU

PL

SE

AR

Factors that can lead to incorrect measurements

Everything that supports the arm during measurement can increase the blood pressure. Make sure that you are in a comfortable position and do not move your arm during the measurement. If necessary, you can place a cushion under the arm to be measured. If the arm artery is lower or higher than the heart this will lead to an incorrect measurement result.

Please note:

Only use arm cuffs that are permitted by the manufacturer and clinically approved! A loose arm cuff or a defective arm cuff in which the bladder is exposed causes incorrect measurement values.

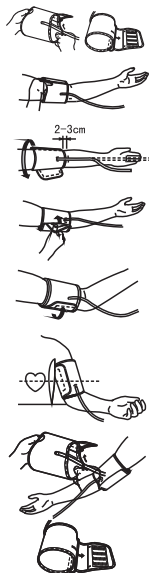
In the case of repeated measurements, blood accumulates in the arm arteries, which can lead to incorrect measurement values.

Continuous blood pressure measurements should be repeated no sooner than after a 1 minute break.

Here it is important that the arm is held upward to ensure that the accumulated blood can flow away.

Putting on the arm cuff

1. Place the arm cuff flat on a table with the hook and loop side facing down. Pass the end of the arm cuff through the metal loop so that a circle is formed. The hook and loop fastener is now visible from the outside.
2. Push the arm cuff over the left upper arm so that the tube lies in the direction of the lower arm.
3. Push the arm cuff over the arm as shown. Make sure that the lower edge of the arm cuff lies about 2 to 3 cm above the elbow and the rubber tube of the arm cuff lies on the inner side of the arm.
4. Pull on the free end of the arm cuff and close the arm cuff using the hook and loop fastener.
5. The arm cuff should now be tight on your upper arm, so that 2 fingers fit between the arm cuff and upper arm. Each item of clothing that is tight on the arm must be removed.
6. Fasten the arm cuff with the hook and loop fastener in such a way that it fits comfortably and not too tightly. Lay your arm on a table (palm upward), so that the arm cuff is at the same height as the heart. Make sure that the tube is not kinked.

Please note:

If it is not possible to put the arm cuff on the left arm, it can also be placed on the right side. All measurements should, however, be carried out on the same arm.

DE

EN

FR

IT

ES

NL

RU

PL

SE

AR

Measurement

After the arm cuff has been suitably positioned you can start the measurement.

1. Press the START/STOP button (A) and all symbols appear on the display. The indicator 0 flashes. The pump now begins to inflate the cuff and the increasing pressure in the cuff is shown on the display.

2. After the suitable pressure has been reached, the pump is stopped and the pressure falls gradually. The arm cuff pressure is shown on the display. In the event that the inflation is insufficient, the pump starts to achieve a higher pressure independently by further inflation pumping.
3. As soon as the device can measure a heartbeat, this is indicated by the flashing of the pulse rate indicator (4) .
4. Once the measurement is completed, the systolic blood pressure, diastolic blood pressure, and the pulse are shown on the display.

5. You can read the measured values on the display until you switch off the device. If no button is pressed for a period of 3 minutes, the device switches itself off automatically to save electricity.

Please note:

The symbol  is shown together with the measured values if an irregular heartbeat is ascertained during the measurement.

Interrupting a measurement

If for some reason (e.g. feeling unwell) it is necessary to interrupt the measurement of the blood pressure, this can be done at any time by pressing the START / STOP button (A). This causes the arm cuff pressure to be reduced immediately and the measurement to be interrupted.

Data storage of the measurement results

This blood pressure meter saves 90 measurement results per user. As soon as there are more than 90 measured values per user, the oldest entry is deleted.

DE

EN

FR

IT

ES

NL

RU

PL

SE

AR

OPERATING THE UNIT

Reading the memory

If the device has been switched off, press the save button "MEM" (C) and the last 3 average measured values will be shown on the display **RL**. The last measured values are shown by pressing the save button "MEM" (C) once more. At every press, the memory moves to the older measured value.

Deletion of the memory

If you are sure that you want to delete all the saved measured values of the chosen user, press the "SET" button (B) 6 times until CL appears in the display. Now press the START / STOP button (A) and CL will flash 3 times in order to delete all saved measurement results. Now press the save button "MEM" (C) and M and "no" will be shown on the display. This means that there are no measurement results in the memory.

SAFETY INSTRUCTIONS

Special warnings!

- People with arrhythmia, diabetes, circulatory disorders or apoplexy problems must only use the device under medical supervision.
- Contact your doctor for specific information on your blood pressure. Self-diagnosis and self-treatment based on the measurement results can be dangerous. Always follow the instructions of your doctor.
- It is not permitted to make changes to this device.
- Do not change anything on this device without the manufacturer's approval.
- If this device has been changed, an appropriate inspection and check must be carried out in order to ensure that the device can be safely used.
- Do not use a mobile phone close to the device. This can lead to failure of the device.
- Please avoid proximity to radiating surfaces, in order to avoid distortion of the measurements.
- Never place the arm cuff tube around the neck as this can lead to choking.
- Only use approved arm cuffs, because otherwise the correct functioning of the device cannot be guaranteed.

Pay attention to the following before first time operation:

- Only for home use.
- Scrupulous supervision is required when the device is being used in the vicinity of children.
- Never place or use the unit in a wet or moist environment.
- Do not use the unit under water, e.g., under the shower.
- Never place the unit in direct contact with fire, gas or oxygen, or hot objects such as hot plates.

SAFETY INSTRUCTIONS

- Take every precaution possible to ensure that the unit does not fall down or get damaged in any other way.
- If problems do arise with the unit, take it immediately to get it repaired.
- Do not lubricate or wash the unit.

Danger!

- Do not place the unit in direct contact with water or other fluids.
- It is not permitted to use the unit outdoors.
- Never touch the unit with wet hands.
- Do not store the unit near a wash basin or bath tub as there is the danger that the unit may fall or be pulled into the wash basin or the bath tub.

Warning!

- Never leave the unit unattended when children or inexperienced persons are in the vicinity of the unit.
- Make sure that children do not play with the device.
- Only use the unit for the applications described in this manual.
- This unit is not designed for use by people (including children) who have limited physical, sensory, or intellectual abilities or who do not have sufficient experience and/or sufficient knowledge. This does not apply if these people are supervised by a person responsible for their safety or receive instructions from them on how to use the unit.
- Do not use the unit underneath blankets or cushions as excessive heat may cause fire, injury or an electrical shock.
- It is not permitted to use the unit outdoors.
- Never let children play with the packaging material; there is a risk of suffocation.
- Never mix old and new batteries.
- Never place heavy objects on the device.
- Do not make any sharp bends in the tube.

TROUBLESHOOTING

Error displays

The following symbols can appear on the display if a measurement is not correct.

SYMBOL	Cause	CORRECTION
E-1	Signal too weak or sudden change in pressure	Put the arm cuff on correctly.
		Re-measure correctly.
E-2	Strong external interference	Remove any possible sources of radiation or mobile phones.
		Keep calm and do not speak during the measurement.

DE

EN

FR

IT

ES

NL

RU

PL

SE

AR

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E-3	Errors seem to occur during inflation	Put the arm cuff on correctly.
		Check whether the tube connecting piece is correctly inserted in the device.
		Check that there are no holes in the tube or the cuff and repeat the measurement
E-5	Abnormal blood pressure	Relax for at least 30 minutes, then re-measure. If you get the same results, you must see your doctor immediately.
	Batteries flat	Replace the old batteries with new ones.

Troubleshooting

Problem	Possible cause to be checked	Solution
No electricity	Check the batteries	Replace with new ones
	Check that the batteries have been correctly inserted (polarity)	Insert batteries correctly
The device does not pump	Check whether the tube connecting piece is correctly inserted in the device	Insert the tube connecting piece gently into the connection
	Check whether the tube connecting piece is intact	Replace with a new tube connecting piece
Error and/or interruption of the measurement	Check whether the arm is laying still and relaxed	Ensure a calm and relaxed position
	Was there talking during the measurement?	Do not speak during the measurement
The arm cuff does not fill with air	Check that the arm cuff is on properly	Fasten the arm cuff correctly
	Check whether the arm cuff or tube has a hole	Replace the arm cuff with a new one

If the fault cannot be rectified, contact the manufacturer before you try to repair the device yourself.

CARE, MAINTENANCE AND STORAGE

- Clean the device with a soft dry cloth.
- Do not use any abrasives or volatile cleaning agents.
- Do not dip the device or parts of it in water.
- Wipe all surfaces of the blood pressure meter and of the arm cuff thoroughly, and make sure that you clean the inner side and outer side of the arm cuff. Take care that no moisture gets into the main device.
- Wipe carefully with a dry cloth in order to remove excess moisture that may still be on the blood pressure arm cuff. Lay the arm cuff out in your home in a rolled-out position so that it can air-dry.
- Do not place the unit in direct contact with water or other fluids.
- Switch the unit off when it is not in use.
- Before putting the **Promed PBM-4.2** into storage for a longer period of time, remove the batteries from the compartment. Leaking batteries can damage the unit.
- Keep the device and its accessories safe in a cool, dry place.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- You can clean your **Promed PBM-4.2** by wiping it over carefully with a cloth moistened with mild soapy water. You may also use isopropyl alcohol or a soap solution. Household detergents and cleaning products are not suitable.
- The unit must not be returned to your dealer or the manufacturer for inspection and recalibration.
- If you should experience other problems, consult your dealer, returning the unit if necessary. Never attempt to repair a fault yourself.

DISPOSAL

Power tools, accessories, and packaging should be recycled in an environmentally suitable manner.

Only for EU countries:

Do not throw power tools in with the domestic waste! In accordance with the European Directive 2012/19/EU on waste electrical and electronic equipment and its implementation into national law, nonfunctioning power tools must be collected separately and recycled in an environmentally suitable manner.

This symbol indicates that disposal of this product in the domestic waste is not permitted within the EU. Waste devices contain valuable recyclable material that should be recycled. Recycling also prevents uncontrolled waste disposal from damaging the environment and human health. Therefore, please dispose of all waste devices using the appropriate collection systems or send the device to the place of purchase for disposal. The place of purchase will then recycle the device.

Rechargeable batteries/batteries:

Do not throw your rechargeable batteries/batteries in the domestic waste, in fire, or in water. Rechargeable batteries/batteries should be collected, recycled, or disposed of in an environmentally friendly manner.

Only for EU countries:

According to the directive 91/157/EEC, defect or used rechargeable batteries/batteries must be recycled. Waste rechargeable batteries/batteries can be directly handed to:
Promed GmbH, Lindenweg 11, D-82490 Farchant.

DE

EN

FR

IT

ES

NL

RU

PL

SE

AR

WARRANTY CONDITIONS

This product was produced with the utmost care and was thoroughly tested before it left the factory. Therefore, we provide a warranty period of 36 months from the date of purchase for this product. **Promed** products conform to their description and the relevant specifications; it is your responsibility to ensure that the product which you have purchased is designed for your intended use.

- In the event of material defects and/or manufacturing errors which appear during proper use and are detected during the warranty period, we will replace all defective parts of the product free of charge within the warranty period, including the labour costs for warranty repairs.
- **The warranty does not cover:**
 - Normal wear on the product
 - Defects which were caused during transportation or storage of the product
 - Defects or damage which were caused through improper use or inadequate maintenance
 - Damage resulting from disregarding the notes in the operating manual
 - Damage resulting from modifications to the product which were not carried out by **Promed**.
 - Damage caused by sharp objects or as a result of torsion, compression, being dropped, an abnormal impact or other actions which are beyond **Promed's** reasonable control.
 - Parts which are subject to wear (e.g. moving parts such as ball bearings, etc., fasteners, etc.) are generally excluded from the warranty.
- **A claim under the warranty is not enforceable if:**
 - the product is not sent back in its original packaging or in adequately secure packaging;
 - it has been modified or repaired by a person or company other than **Promed** or one of **Promed's** authorised dealers;
 - the product has been repaired using parts which are not approved by **Promed**;
 - the serial number / LOT number of the product has been removed, deleted, changed or made illegible.
- For reasons of hygiene, any products which have been exposed to direct contact with a body or with bodily fluids (e.g. blood) must be packaged in an additional plastic bag before they are sent back. In this case, a specific note of this situation must be made in the package or in the accompanying documents.
- For components or products which are replaced during the warranty period, a warranty is only provided for the remainder of the original warranty period; provided that the product or part replacement was carried out by **Promed** or one of **Promed's** authorised dealers.
- The warranty period begins on the day of purchase. Claims under the warranty must be established within the warranty period. Complaints arising after the expiry of the warranty period cannot be considered.
 - Under these warranty conditions, the warranty shall only apply if the date of purchase can be proven by means of a sales receipt or similar.
 - We reserve the right to make technical and visual changes as well as changes to the features.
- This warranty is only valid and enforceable in the country in which it was purchased by the original buyer, provided that it was **Promed's** intention that the product should be offered for sale in this country. This warranty is also enforceable in any country within the European Economic Area in which **Promed** has an authorised importer or sales partner. Depending on each country, specific and differing warranties and guarantees may be applicable owing to the particular applicable legislation. These legal provisions are neither excluded nor limited by these warranty conditions. To the extent permissible under national law, the warranty period shall not be extended, renewed or otherwise affected by any subsequent resale, repair or replacement of the product.
- The provisions of the UN Convention on Contracts for the International Sale of Goods shall not apply.
- Our warranty conditions do not affect the seller's legal warranty obligation.
- To the fullest extent possible under the applicable mandatory legislation, these warranty conditions constitute your sole and exclusive legal remedy and shall apply in place of any other express or implied warranty conditions. **Promed** is not liable for any unusual, incidental, punitive or consequential damages including, but not limited to loss of profits, loss of use, loss of revenue, costs of replacement equipment or facilities, third party insurance claims, and damage to property which are the result of the purchase or use of the product or occur owing to a breach of warranty, a breach of contract, negligence, product defects or any other legal or statutory circumstances, even though **Promed** was aware of the possibility of such damages. **Promed** is not liable for any delay in warranty claims.
- **Promed** cannot be held liable for any possible translation errors.

The following information is essential for smooth processing:

1. Original proof of purchase/receipt or dealer's stamp with date of purchase
2. Defect discovered
3. Product designation / type / serial/batch number

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SHENZHEN URION TECHNOLOGY CO.,LTD.
Floor 4-6th of Building D, Jiale Science & Technology Industrial Zone,
No.3, ChuangWei Road, Heshuikou Community, MaTian Street,
GuangMing New District, 518106 ShenZhen, PEOPLE'S REPUBLIC OF CHINA

EU BEVOLMÄCHTIGTER | EU AUTHORISED REPRESENTATIVE | MANDATAIRE POUR L'UE | RAPPRESENTANTE AUTORIZZATO UE | ORGANISMO AUTORIZADO DE LA UE | EU-GEVOLMACHTIGDE | УПОЛНОМОЧЕННЫЙ ЕС | РЕНОМОСНИК UE | EU:S REPRESENTANT | وكيل الاتحاد الأوروبي

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

IMPORTEUR | IMPORTER | IMPORTATEUR | IMPORTATORE | IMPORTADOR | IMPORTEUR
ИМПОРТЕР | IMPORTER | IMPORTÖR | المستورد

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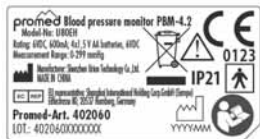
Symbol for **WARNING!** Indicates sections of the user manual that are safety-relevant!

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Pay attention to the symbol for the user manual!

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Symbol for Manufacturer and manufacturing Factory

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Symbol for Authorized representative in the European Community

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EMC DECLARATION

IEC 60601-1-2:2014 ME EQUIPMENT and ME SYSTEMS identification, marking and documents for Class B product

Instructions for use

The ME EQUIPMENT or ME SYSTEM is suitable for home environment

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation."

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment (detail model name), including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

If any: a list of all cables and maximum lengths of cables (if applicable), transducers and other ACCESSORIES that are replaceable by the RESPONSIBLE ORGANIZATION and that are likely to affect compliance of the ME EQUIPMENT or ME SYSTEM with the requirements of Clause 7 (EMISSIONS) and Clause 8 (IMMUNITY). ACCESSORIES may be specified either generically (e.g. shielded cable, load impedance) or specifically (e.g. by MANUFACTURER and EQUIPMENT OR TYPE REFERENCE).

If any: the performance of the ME EQUIPMENT or ME SYSTEM that was determined to be ESSENTIAL PERFORMANCE and a description of what the OPERATOR can expect if the ESSENTIAL PERFORMANCE is lost or degraded due to EM DISTURBANCES (the defined term "ESSENTIAL PERFORMANCE" need not be used).

Technical description

1. all necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life.

2. Guidance and manufacturer's declaration -electromagnetic emissions and Immunity.

Table 1

Guidance and manufacturer's declaration - electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not application

Table 2

Guidance and manufacturer's declaration - electromagnetic Immunity		
Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Power supply lines: ±2 kV input/output lines: ±1 kV	Not application
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV. line(s) to earth: ±2 kV. 100 kHz repetition frequency	Not application

EMC DECLARATION

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % 1 cycle And 70% 25/30 cycles Single phase: at 0 0 % 300 cycle	Not application
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands) 80 % Am at 1kHz	Not application
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE UT is the a.c. mains voltage prior to application of the test level.		

Table 3

Guidance and manufacturer's declaration - electromagnetic Immunity							
Radiated RF IEC61000-4-3	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
(Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380 – 390	TETRA 400	Pulse modulation 18 Hz	1,8	0.3	27
	450	430 – 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
	710 745 780	704 – 787	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0.3	9
	810 870 930	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
	1720 1845 1970	1700 – 1990	GSM 1800; CDMA 1900; GSM 1900; DECT;	Pulse modulation 217 Hz	2	0.3	28
	2450	2400 – 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
	5240 5500 5785	5100 – 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9

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Shanghai International Holding Corp.GmbH (Europe)
Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0