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- N.B. Only use equipment specifically designed for electro-stimulation with a low current intensity.**
- Before switching on the electro-stimulator, check the perfect adherence of the electrode to the skin surface, over its entire area. It may be useful to moisten the skin before applying the electrodes to improve adhesiveness. Do not use the product if the contact between the electrode and the skin is incomplete or unstable.

• If the stimulation capacity is lower than normal, do not increase the output, without having previously checked that the entire surface of the electrode is in contact with the patient's skin.

- Interrupt the treatment if the patient complains discomfort or a burning sensation.
- After the session, turn off the electrostimulator, disconnect the cable from the electrode and gently remove the electrode by lifting it up by one corner.

N.B. Rapid removal of the electrode could cause skin traumas. Do not remove by pulling the connection part of the cable.

WARNING

- The product is supplied Not Sterile.
- This is a single-use product, do not re-use it. Reusing the product may lead to: alteration of materials; loss of initial functional features of product.
- The device must be connected and started up by qualified personnel.
- Do not apply electrodes on skin affected by abrasions, erythema or wounds.
- Do not use the electrodes after the "use before" date indicated on the label.
- Keep the packet closed in order to prevent the unused electrodes from becoming dehydrated.
- Do not re-position the electrode.

WARNING: reduced contact between the skin and the electrode can concentrate the current in one point and cause burns.

CONTRA-INDICATIONS The product must not be used: • With pregnant women. • With epileptic patients. • With wearers of pacemaker.

• In subjects with progressive episodes of skin irritation or contact allergies.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pad's label. Putting external heavy weights on the package, the product could be damaged.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force.

GENERAL NOTES

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

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