promed tens 1000 s

Model: WL-2103A

Gerät für die Transkutane Elektrische Nerven-Stimulation (TENS)
Transcutaneous Electrical Nerve Stimulation unit (TENS)
Appareil d'électro-neurostimulation transcutanée (TENS)
Apparecchio per l'elettrostimolazione nervosa per via transcutanea (TENS)
Estimulador eléctrico transcutáneo de nervios (TENS)
Apparaat voor Transcutane Elektrische Zenuwstimulatie (TENS)
Urządzenie do przeskórnego elektrycznego stymulowania nerwów (TENS)
Аппарат для чрескожной электронейростимуляции (ЧЭНС)
Transkutanöz elektriksel sinir stimülasyonu (TENS) cihazı

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

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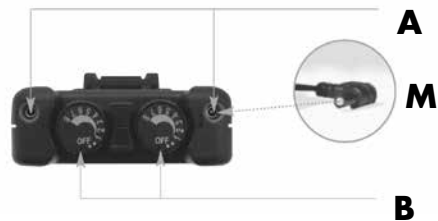
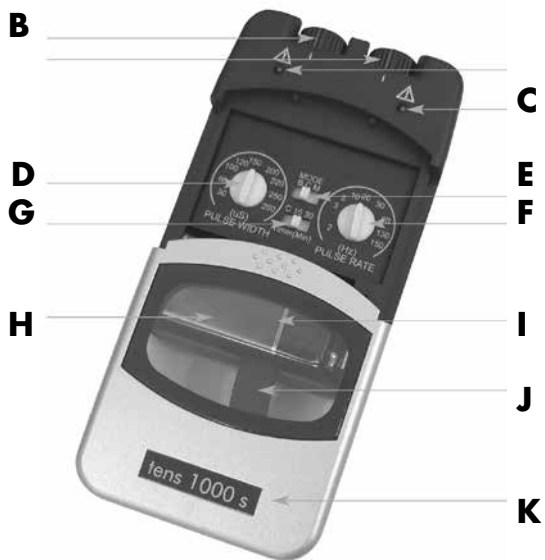
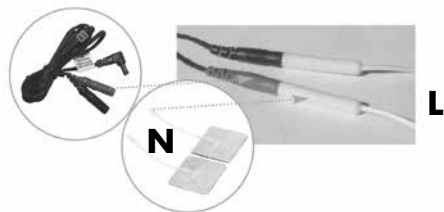
TR

AR

- A.** Sockets for the electrode leads (left and right channel)
- B.** Intensity regulators (left and right channel)
- C.** Status LEDs
- D.** Pulse width regulator
- E.** Mode selector
- F.** Pulse frequency regulator
- G.** Timer
- H.** Battery
- I.** Battery compartment
- J.** Battery strap
- K.** Cover
- L.** Electrode plugs
- M.** Lead plug (the same for both channels)
- N.** Electrodes

GETTING STARTED	p. 22	MANUFACTURER	196
CONTENTS	p. 23	TYPE PLATE	196
SCOPE OF DELIVERY / ACCESSORIES	p. 24	CERTIFICATION NUMBER	196
DESCRIPTION OF THE UNIT	p. 25	NORMALIZED SYMBOLS	182/183
INTRODUCTION	p. 26/27	CE DIRECTIVES	184/185
WARNINGS AND PRECAUTIONS	p. 28-32	TECHNICAL SPECIFICATIONS	186/187
OPERATING THE UNIT	p. 33-36	TRANSPORT/STORAGE/OPERATING CONDITIONS	188/189
PLACEMENT OF ELECTRODES	p. 37	ACCESSORIES	190-193
APPLICATION EXAMPLES	p. 38	CLINICAL RECOMANDATIONS	194/195
TROUBLESHOOTING	p. 39	GUARANTEE CARD	197
CARE, MAINTENANCE AND STORAGE	p. 39		
DISPOSAL			
GUARANTEE	p. 40		

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

Scope of delivery:

- Device
- Transport case
- Electric cable
- Fabric electrodes, 40 x 40 mm
- Instruction leaflet
- Instruction leaflet, fabric electrodes
- 9 Volt block battery

Not included as accessories / optionally available:

- Rechargeable battery
- Battery charger

Please read the user manual thoroughly before using the device for the first time.

Thank you for purchasing the **Promed tens 1000 s**. You have purchased a high-quality product designed for personal hygiene and wellbeing purposes. **Promed** is a leading company with decades of experience in personal hygiene, wellness and health areas.

The **Promed tens 1000 s** unit has been designed and manufactured in accordance with the Medical Device Directives 93/42/EEC to guarantee quality during use. The device may be used once this user manual has been read.

We, as the manufacturer, cannot be made liable in any way for injury or damages to people or objects that arise from failure to comply with this user manual. We wish you a lot of enjoyment with your new **Promed tens 1000 s**. We would like to familiarize you with the therapy unit in the following sections. The **Promed tens 1000 S** device is only intended for domestic use. Please read the user manual thoroughly before using the device for the first time.

You can receive professional advice wherever **Promed** products are sold or get in touch with us if you have any questions. We can provide you with the name of the representative responsible for you.

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Further informations can be found on our homepage:
www.promed.de

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

Your **tens 1000 s** is a battery-powered device with two adjustable output channels. The TENS unit generates electrical impulses. You can use the unit's controls to set and vary the amplitude, duration and modulation of these impulses. The TENS unit's controls are very straightforward to use, while the safety cover stops your settings being modified unintentionally. The intensity level is adjustable according to the needs of patients. Before using, please read all the instructions in this user manual carefully and keep it safe for future reference.

What is pain?

Pain is the body's own early warning system. Feeling pain is important, as it indicates an abnormal condition within the body and serves as a warning to us before any further damage or injury occurs. However, long-term, persistent pains – often known as chronic pain – serve no obvious useful purpose once a diagnosis has been made. TENS was developed to soothe or eliminate certain types of chronic and acute pain.

We differentiate between two types of pain:

- **Acute pain**

as a chief symptom can often help the doctor with diagnosis and the acute pain has a protective function for the patient.

- **Chronic pain**

can often become part of the illness in itself. A patient suffering from chronic pain will often suffer for years and experiences changes in his/her personality structure.

General pain analysis

1. Heller, pointed or stabbing pain

Apply frequencies < 50 Hz to < 20 Hz

Select pulse width as high as possible > 150 μ sec

2. Dull, throbbing pain

Apply frequencies > 50 Hz (usually)

Pulse width as wide as possible (throbbing and/or pain must not be intensified), which means usually small pulse widths < 120 μ S

If possible, no muscle tissue should be stimulated, as otherwise this might cause aching muscles!

EXPLANATION OF TENS

Transcutaneous Electrical Nerve Stimulation (TENS) is a non-invasive, drug free method of controlling pain. TENS uses tiny electrical impulses sent through the skin to nerves to modify your pain perception. TENS does not cure any physiological problem; it only helps control the pain. TENS does not work for everyone; however, in most patients it is effective in reducing or eliminating the pain, allowing for a return to normal activity.

How does TENS work?

Promed tens 1000 s works by passing harmless electrical signals into the body from the electrode pads. This relieves pain in two ways:

- Firstly, it blocks the body's pain signals which are normally transmitted from the area of damage through the nerve fibres to the brain - TENS interrupts these pain signals.
- Secondly, TENS stimulates the body's production of endorphins – its own natural painkillers.

Possible TENS applications

Generally the **Promed tens 1000 s** shall be used for the following medical indications or the following complaints as a TENS unit for treatment:

- For symptomatic relief of chronic intractable pain
- For post traumatic pain (acute pain occurring)
- For post surgical pain (pain caused by surgery)

Indications:

- Myalgia (muscle pain), approx. 100–120 Hz
- Muscle relaxation, approx. 10 Hz
- Muscular pump (regulate the intensity on both channels alternatingly high and low), approx. 50 Hz
- Calming of the sympathetic trunk (reducing the stress on the sympathetic trunk), approx. 120 Hz
- Stimulation of the resorption (supporting the reuptake performance), approx. 100 Hz
- Joint pain, approx. 50–100 Hz, approx. 2–4 Hz
- Mackenzie's zones (muscular stimulation points), approx. 120 Hz
- Head's zones (sensitive to painful skin zones), approx. 120 Hz
- In the case of sensitive persons, pulse width < 150 µs
- In the case of thick muscle groups or sportspeople, pulse width > 150 µs

THERAPY RECOMMENDATIONS TENS

Therapeutic goal	Frequency	Alternative
Stimulation of the skin	approx. 50 Hz	approx. 1 - 4 Hz
Analgesia (pain relief)	approx. 50 - 180 Hz	approx. 2 - 4 Hz
Detonisation (relaxing, slackening)	approx. 100 - 120 Hz	approx. 2 - 8 Hz
Trophism (Nutrition of the tissue)	approx. 100 - 120 Hz	—

WARNINGS AND PRECAUTIONS

Warnings

- If you are under the care of a Physician, consult with your Physician before using this System.
- The long-term effects of this System are not known.
- Do not place the pads on or close to your heart.
- Do not place the pads around or close to your neck. Do not apply stimulation over the neck. Severe spasm of the muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing. Stimulation over the neck could also have adverse effect on hearing or blood pressure.
- Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart.
- Do not place the pads on or around your head. The effects of stimulation of the brain are unknown.
- Do not use the pads over or close to sores.
- Do not place the pads on the front or sides of the neck across or through the heart (one pad on the front of the chest and one on the back), in the genital region, or on the head, because of the risk of stimulating inappropriate muscles and organs.
- Do not place the pads over any recent scars, broken or inflamed areas of infection or susceptibility to acne, thrombosis or other vascular problems (e.g. varicose veins), or any part of the body where feeling is limited.
- Do not place the pads over areas of injury or restricted movement (e.g. fractures or sprains).
- Do not use this System while sleeping.
- Do not use if you feel numbness.
- Do not use this System in or close to water.
- Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart, which could be lethal.
- Do not use the pads over or close to cancerous lesions.
- Use the pads only on normal, healthy, clean and dry skin. Do not use the pads on open wounds or rashes, or over swollen, red, infected or inflamed skin.
- If you have ever had back surgery, consult your Physician before using this System.
- Electronic monitoring equipment (such as ECG and ECG alarms) may not operate properly when electrical stimulation is in use.
- You must position the pads and operate the unit **ONLY** as indicated in this manual.
- Avoid areas in injury or restricted movement (e.g. fractures or sprains)
- Avoid placing the pads over metal implants.
- Do not use in the bath or shower, or in an environment of elevated humidity (e.g. Sauna, hydrotherapy, etc).
- Operation in close proximity, such as 3 feet (1 meter), to shortwave or microwave therapy equipment may produce instability in the device output and may shut the device off.
- Do not use the device in an environment where flammable or explosive fumes may exist.
- Patient should never operate potentially dangerous machinery such as power saws, automobiles, etc. during electrical stimulation. It might result in burs at the site of the stimulator electrodes and possible damage to the stimulator if you connect to high frequency equipment.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

WARNINGS AND PRECAUTIONS

- Application of electrodes near the thorax may increase the risk of cardiac fibrillation.
- The system is internally powered (power form battery pack), IP 22, continuous operation.
- Potential hazard from simultaneous connection of a patient to a high frequency surgical ME equipment and the stimulator that may result in burns and possible damage to the stimulator.
- Only use this device in the specified operating condition
- The safety of neuromuscular stimulation during pregnancy has not been established.
- **Use caution when/if:**
 - Sensory nerve damage is present by a loss of normal skin sensation.
 - Use caution prior to using this device on patients suspected of having heart disease.
 - Use caution for patients with suspected or diagnosed epilepsy when using this device.
 - Use caution following recent surgical procedures when muscle contraction may disrupt the healing process.
 - Use caution when there is a tendency to hemorrhage, such as following acute trauma or fracture.
 - A menstruating or pregnant uterus.
 - Patient experiences skin irritation due to electrical stimulation or the electrical conductive medium used, remove the electrodes, discontinue stimulation, and consult the clinician. Irritation may be reduced by an alternative conductive medium or an alternative electrode placement. Isolated cases of skin irritation may occur at the site of electrode placement following long term application.

WAIT BEFORE USING THE UNIT :

- At least 6 weeks after the birth of your baby (you must consult your doctor before use).
- One month after an IUD contraceptive device (e.g. coil) has been fitted (you must consult your doctor before use).
- At least 3 months after having a caesarean section (you must consult your doctor before use).
- The heavy days of your period have finished, because vigorous abdominal exercise is not recommended at this time.

PRECAUTIONS

- Read User Manual before using this System for the first time.
- Keep this manual available whenever you use your System.
- The System is intended for personal use on healthy adult muscle only.
- The safety of using the System during pregnancy or birth has not been established.
- The effectiveness of the System depends greatly on a person's individual physical condition. It may not always be effective for every user.
- Place electrodes in accordance with illustrations in the User Manual.
- This unit should not be used while driving, operating machinery or during any activity in which involuntary muscle contractions may place the user at undue risk of injury.
- Some users may experience skin irritation or hypersensitivity due to the electrical stimulation or the conductive medium.
- Keep this device out of the reach of children. If the patient is a child, make sure he/she is properly supervised during electrical stimulation. Strangulation resulting from baby or child entanglement in monitoring cables.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

WARNINGS AND PRECAUTIONS

- Application of moderate heat (thermal wrap) to muscles as well as moistening skin prior to treatment improves treatment efficacy; use of cold packs on treated muscles after treatment is likewise recommended.
- This unit should only be used with the leads, electrodes and accessories provided by the manufacturer.
- The device is not intended for medical use, for the treatment of any medical condition or for any permanent physical changes.
- Contact Well-Life, or an authorized dealer, if your unit is not working correctly. Do not use in the meantime.
- An effective session should not cause discomfort.
- For first time users, muscle stimulation can be an unusual sensation. We recommend that you begin in a seated position with low stimulation intensity settings to familiarize yourself with the sensation before progressing to higher intensity settings.
- The leads and pads must not be connected to other objects.
- Do not over exert yourself while using muscle stimulation. Any workout should be at a comfortable level for you.
- Do not place pads over jewelry or body piercings.
- If you have a tendency to bleed internally following an injury.
- If you recently had surgery, or have ever had surgery on your back.
- If areas of skin lack normal sensations, such as skin that tingles or is numb.
- PATIENT with an implanted electronic device (for example a cardiac pacemaker) should not be subjected to stimulation unless specialist medical opinion has first been obtained
- During menstruation or during pregnancy.
- Some people may feel skin irritation or experience a very sensitive feeling in the skin due to electrical stimulation. If this occurs, stop using your System and consult your Physician.
- If skin under one of more pads feels irritated after using the stimulator for a long period of time, use the stimulator for a shorter period of time.
- Minor redness at stimulation placement is a normal skin reaction. It is not considered as skin irritation, and it will normally disappear within 30 minutes after the electrodes are removed. If the redness does not disappear after 30 minutes from the removal of electrodes, do not use the stimulator again until after the excessive redness has disappeared.
- Turn off the stimulator if the stimulation feels unpleasant or does not provide pain relief.
- Keep your System out of the reach of children.
- Use your stimulator only with the pads, snap cables and accessories recommended by the manufacture.
- Do not use this System when driving, operating machinery or when swimming.
- Before removing the pads, be sure to power off device to avoid unpleasant stimulation.

Use Caution and consult Physician before using System if any of the following conditions apply to you:

- You have any serious illness or injury not mentioned in this guide.
- You have recently undergone a surgical procedure.
- You take insulin for diabetes.
- You use the unit as part of a rehabilitation program.
- If you have suspected or diagnosed heart problem.
- If you have suspected or diagnosed epilepsy.

WARNINGS AND PRECAUTIONS

After strenuous exercises or exertion:

- Always use lower intensity to avoid muscle fatigue.

Important :

- Effectiveness is highly dependent upon patient selection by a clinician qualified in the management of pain or rehabilitation.
- Do not use your unit at the same time as any other device, which transfers an electrical current into the body (e.g. another muscle stimulator).
- Cease using your unit if you are feeling light headed or faint. Consult doctor if this happens.
- Do not touch the pads or metal studs while the unit is switched on.
- Do not use unit if you are wearing a belly button ring. Remove ring before session.
- Use the device with only the leads and electrodes provided for use by the WellLife with your device. Any others may not be compatible with your unit and could degrade the minimum safety levels. Use only the electrode placements and stimulation settings prescribed by your practitioner.
- This device is for external use only.
- Choking resulting from a child swallowing a small part that has become detached from the device.

Note: If you are in any doubt about using device for any reason, please consult your doctor.

Pad/Electrode Precautions

- To reposition the pads during a session, always pause the program currently running, reposition the pads and then restart the program.
- The pads are for single person use only.
- Do not plunge the pads into water.
- Do not apply solvents of any kind to the pads.
- Always ensure the unit is OFF before removing the pads.
- Apply the whole surface of the pads firmly to the skin. Do not use pads, which do not adhere properly to the skin.
- If your skin is red under the pad after a session, do not start another session in the same area until your redness has completely disappeared.

Adverse Reactions

- You may experience skin irritation due to prolonged exposure to the stimulation electrodes applied to your skin.
- You may experience potential allergic reactions to accessible materials used in the electrodes.
- You may experience headache and other painful sensations during or following the application of electrical stimulation near your eyes and to your head and face.
- You should stop using the device and should consult with your physician if you experience adverse reactions from the device.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

**Conditions that may affect your TENS/EMS**

Since the stimulator is a battery-operated electronic System, its output performance and safety may be affected greatly in extreme humidity. Therefore, it is very important to keep the system device(s) dry to ensure the safety and performance of the stimulator.

- The patient may also be an intended operator, at least 15 years old and 8 years intensive reading experience(school), no maximum, and can safely change the batteries.
- This system is for home use, indoor, not intended for professional use
- This system is for daily use, no operation time limit on device but it is recommended not exceeding 60 minutes per day.
- If there is any other problem, please consult or return the device to your distributor. Do not try to repair a defective device.
- The effects of degraded sensors and electrodes, or loosened electrodes, which can degrade performance or cause other problems.
- For plug-in equipment , the power outlet socket must be located near the equipment and must be easily accessible
- **WARNING:** No modification of this system is allowed.
- **WARNING:** Other cables and accessories may negatively affect EMC performance .
- **WARNING:** Do not stack and store this system closed to other equipment.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

Before you start:

- The Promed Tens 1000 s device is only intended for domestic use.
- Make sure the batteries are installed correctly.
- Connect the cable to the electrode pads and then connect to the unit.
- Attach the electrode pads to the area of pain.
- Do not use the electrode pads if it is scratched or damaged in any way.

Check/Replace the battery

1. Open the battery compartment cover (I).
2. Insert the battery into the battery compartment. Make sure to insert the battery correctly. The positive and negative poles of the battery must face the corresponding marks in the battery compartment of the device.
3. Recover the battery compartment (II) cover if you want to use it.

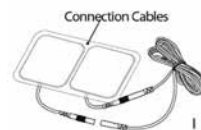
Caution:

1. Remove the batteries if the device is not in use for long periods of time.
2. Warning: If batteries leak and come into contact with the skin or eyes, wash immediately with copious amounts of water.

3. Batteries must be handled by an adult. Keep batteries out of the reach of children.
4. Remove exhausted batteries from the unit.
5. Dispose of batteries safely according to battery manufacturer's instructions.

Connect the electrode pads to the cables:

Hold the electrode plug contacts (L) and insert them into the sockets of the self-adhesive electrodes. Make sure that the bare metal of the electrode pins is not exposed.

Caution:

Always use the cable that supplied with manufacturer or distributor and use the electrode pads with CE mark, or are legally marketed in the US under 510 (k) procedure.

Connect cable to unit

The leads supplied with the TENS unit are to be connected to the sockets on the upper side of the unit (A). Hold the plug by the insulated part and push the end of the plug into one of the sockets. You may use one or two sets of leads. After you have connected the leads to the stimulator unit, connect an electrode to each lead (N). The leads supplied with the TENS unit conform to the FDA Compliance Standards.

**Caution:**

Do not insert the plug of the electrode cable into an AC power supply socket.

Place electrode pads on skin

Apply electrode pads to the area where the pain is felt (refer to Annex A: placement of electrodes). Before applying electrodes, be sure the skin surface over which electrode pads are placed is thoroughly clean and dry. Make sure the electrode pads are pressed firmly to the skin and make good contact between the skin and the electrode pads.

**Caution:**

1. Before applying the electrode pads to the body, ensure the skin surface is clean and free from lotions or moisturizers.
2. Do not switch the device on when self-adhesive electrode pads are not positioned on the body.
3. Never remove the pads from the skin whilst the device is still switched on.
4. For hygiene reasons we recommend that the electrode pads are replaced every 30 days.
5. Please make sure the device is completely turned off before connecting.
6. It is recommended that, at minimum, 40mm x 40mm self-adhering based, square electrodes are used at the treatment area.
7. For hygienic reasons, each patient should use their own set of electrodes.

Turn the device on:**OPERATING THE UNIT**

Safety cover: The controls for pulse width, pulse frequency, mode selection and modulation selection are protected by a safety cover. Press the cover (K) in the top middle in order to push it downwards.

Intensity: The intensity regulators (**B**) are to be found on the upper side of the unit and are used to set the intensity of stimulation. They are also used to switch the unit on and off.

Mode: The mode selector (**E**) is used to set the type of treatment. The three available modes are: burst stimulation [Burst] (**B**), continuous stimulation [Continuous] (**C**) and modulated stimulation [Modulated] (**M**).

Pulse width (μs): The control knob (**D**) regulates the pulse width (= the duration of individual impulses) for both channels.

Pulse frequency (Hz): The pulse frequency knob (**F**) regulates the number of pulses per second for both channels.

Timer: With the aid of the timer control (**G**), you can set the duration of TENS treatment. There are three possible settings: 15, 30 minutes and C (continuous). Use the switch to select the desired setting.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

Resetting the timer: To restart treatment or reset the timer, simply switch the intensity regulator **(B)** into the OFF position, then back to the ON position.

Mode functions: Burst mode **(B)** triggers two individual bursts per second. In this mode, the pulse frequency is set at 100 Hz per second, while the pulse width is adjustable. In continuous stimulation mode **(C)**, impulses are transmitted continuously at the selected settings for intensity, pulse frequency and pulse width. In modulation mode **(M)**, the pulse width set by the user is reduced by 60 %. This lower pulse width is maintained for 2 seconds before returning to the original user setting, which is then maintained for 3.5 seconds. The cycle is then repeated. Intensity and pulse frequency are adjustable by the user.

SELECTION, CARE AND USE OF ELECTRODES

Your doctor should decide which type of electrode is best suited to your treatment. To achieve optimum stimulation results and avoid skin irritation, follow the procedures described in the instructions supplied with the electrodes. The electrode package will include tips and information on care and maintenance and the correct storage of your electrodes.

Using electrodes

Please make sure that the **tens 1000 s** is switched off when connecting or removing electrodes.

Positioning the electrodes

Please refer to the operating instructions for the **tens 1000 s** electrodes included with the package. To ensure that TENS therapy remains effective for a long time, you should replace the TENS electrodes regularly. The skin to be treated should be clean and free of grease.

Handling electrodes

After use, the electrodes must be put back on the protective foil to ensure that they continue to function effectively. If the adhesive properties of the electrode start to deteriorate after being used many times, you may use a contact gel, which you can remove with a cloth after use. It is necessary to replace the electrodes after a maximum of 30 uses.

For hygienic reasons, each patient should use their own set of electrodes.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

CONNECTING THE TENS UNIT**1. Preparing the skin**

Prepare the skin as described above and in the instructions that came with the electrodes. Position the electrodes in the area recommended by your doctor.

2. Connecting the leads to the electrodes

Connect the leads to the electrodes before attaching the electrodes to the skin.

Caution: Make sure that the intensity regulators (**B**) for both channels 1 and 2 are in the „OFF“ position.

3. Attaching the electrodes to the skin

Attach the electrodes to your skin in the manner recommended by your doctor.

4. Connecting the leads to the TENS unit

Plug the lead into the appropriate channel socket, pushing the plug as far as possible into the socket (see “**Connecting the electrode leads**”).

5. Selecting the settings for treatment

Make sure that the settings selected on the unit are those recommended by your doctor.

6. Setting channel intensity

The intensity regulator (**B**) is to be found on the upper side of the unit. Turn the control knob for channel 1 or 2 in a clockwise direction. The power LED will light up when the unit is switched on.

Slowly turn the control knob in a clockwise direction until you reach the intensity setting recommended by your doctor. If both channels are being used, repeat the same procedure for the other channel.

SKIN CARE TIPS

Effective skin care is especially important in order to ensure comfortable use of your tens 1000 s unit.

- Always wash the areas of skin where the electrodes are to be placed using mild soap and water. Thoroughly wash off any soap remains and carefully dry the skin before using an electrode.
- To ensure effective contact between the skin and the electrode, excess hair should be cut but not shaved off prior to treatment.
- You may prepare the skin with a skin care product recommended by your doctor. After applying it, let it dry before attaching the electrodes as described. This will reduce the likelihood of skin irritation and prolong the life of your electrodes.
- Avoid excessive pulling of the skin when attaching the electrodes. The best way to achieve this is to place the electrode on the skin and carefully smooth it out from the centre outwards.
- When removing the electrodes, always pull them off in the direction of hair growth.
- It may be helpful to apply a skin cream to the area being treated while the electrodes are not being worn.

DE

EN

FR

IT

ES

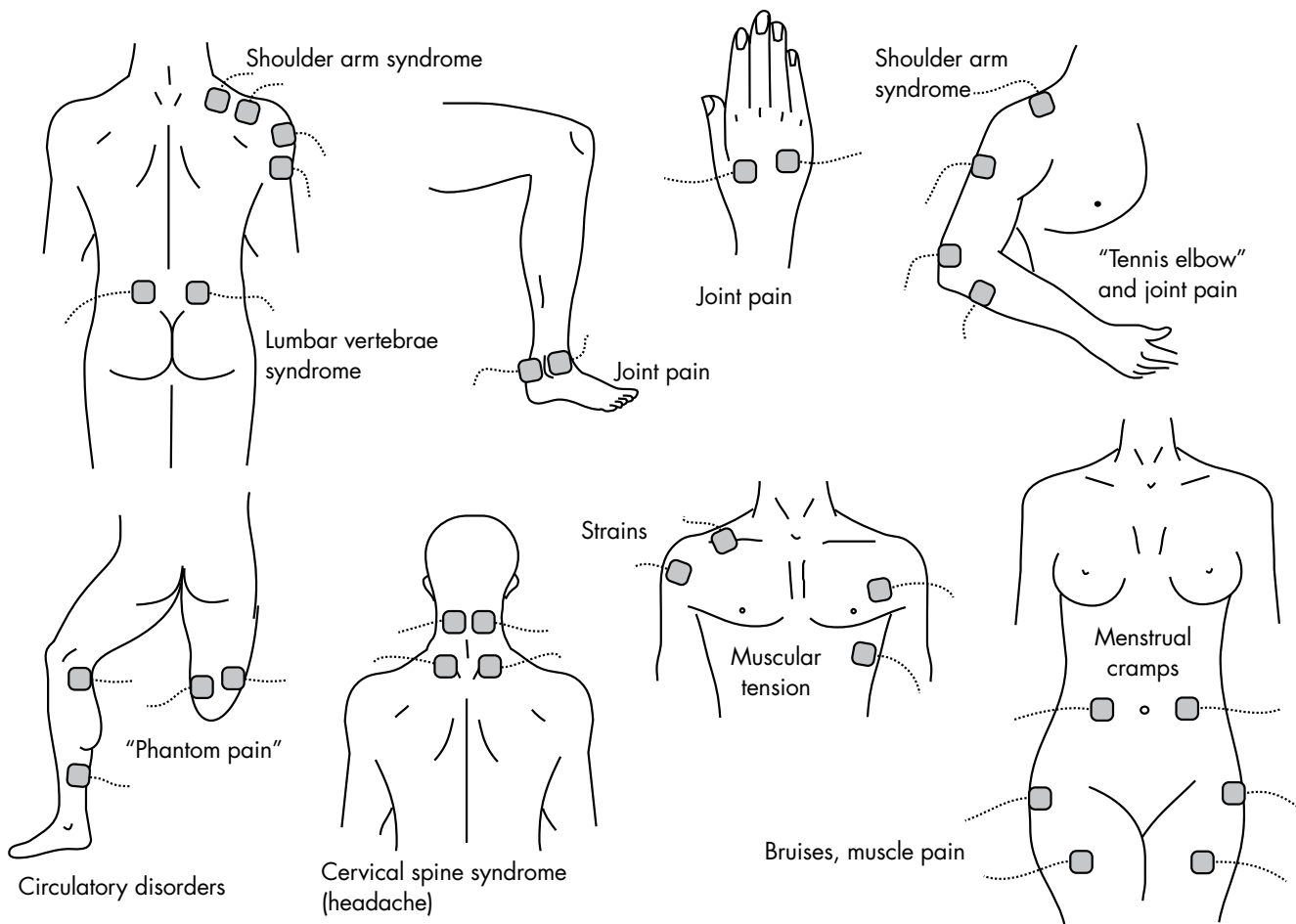
NL

PL

RU

TR

AR

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

APPLICATION EXAMPLES

Mode	Intensity	Pulse width	Pulse rate	Cycle time	Possible areas of use
Continuous	regulierbar 0-80 mA	regulierbar 30-260 μ Sek	regulierbar 2-150 Hz	nicht verfügbar	To ease muscle parts and to stimulate circulation. To ease stabbing and intense pain as well as dull and throbbing pain. To warm, loosen up and relax the limbs (legs, arms) and the back. Counteracts you from getting used to the stimulus. To ease pain and tension. To combat pain in the neck and shoulders and symptoms in the knees.
Burst	regulierbar 0-80 mA	regulierbar 30-260 μ Sek	100 Hz fest 2 Burst pro Sek.	nicht verfügbar	To ease muscle parts and to stimulate circulation. To ease stabbing and intense pain as well as dull and throbbing pain. To warm, loosen up and relax the limbs (legs, arms) and the back. Counteracts you from getting used to the stimulus. To ease pain and tension. To combat pain in the neck and shoulders.
Modulation	regulierbar 0-80 mA	moduliert vom eingestellten Impulsbreiten- Wert um 60% herunter und kehrt zum ursprünglichen Wert zurück	regulierbar 2-150 Hz	5,5 Sek Gesamtzeit	To ease muscle parts and to stimulate circulation. To ease stabbing and intense pain as well as dull and throbbing pain. To warm, loosen up and relax the limbs (legs, arms) and the back. Counteracts you from getting used to the stimulus. To ease pain and tension. To combat pain in the neck and shoulders and symptoms in the knees.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

If your TENS unit is not functioning correctly

1. Make sure that the battery is correctly inserted or replace the battery. When inserting a battery, make sure that the polarity is correct.
2. If the ON/OFF LED is flashing and there is no stimulation, check that the leads are attached correctly and that the electrodes are correctly positioned. If the unit seems to be functioning, yet you still feel no stimulation, it is possible that the electrodes or the leads need replacing.
3. If the battery is charged but the unit is not working, turn both intensity controls (**B**) to the OFF position (counterclockwise) and then slowly back to the ON position.

- Do not place the unit in direct contact with water or other fluids.
- Switch the unit off when it is not in use.
- Before putting the **Promed tens 1000 s** into storage for a longer period of time, remove the batteries from the compartment. Leaking batteries can damage the unit.
- Keep the unit and its accessories in a cool, dry place in the case provided.
- Handle the electrodes by their edges when removing them. To avoid damaging them, never pull at the electrode leads themselves.
- Do not make any sharp kinks in the connecting leads or electrodes.
- After use, stick the electrode pad onto the protective plastic film.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- Never place any heavy objects on the device.
- You can clean your **Promed tens 1000 s** by wiping it over carefully with a cloth moistened with mild soapy water. You may also use isopropyl alcohol or a soap solution. Household detergents and cleaning products are not suitable.
- The unit must not be returned to your dealer or the manufacturer for inspection and recalibration. If you need such inspections or recalibrations because of your internal Quality Management System don't hesitate to contact Promed.
- If you should experience other problems, consult your dealer, returning the unit if necessary. Never attempt to repair a fault yourself.

DE

EN

FR

IT

ES

NL

PL

RU

TR

AR

Power tools, accessories, and packaging should be recycled in an environmentally suitable manner.

Only for EU countries: Do not throw power tools in with the domestic waste! In accordance with the European Directive 2012/19/EU on waste electrical and electronic equipment and its implementation into national law, nonfunctioning  power tools must be collected separately and recycled in an environmentally suitable manner.

This symbol indicates that disposal of this product in the domestic waste is not permitted within the EU. Waste devices contain valuable recyclable material that should be recycled. Recycling also prevents uncontrolled waste disposal from damaging the environment and human health. Therefore, please dispose of all waste devices using the appropriate collection systems or send the device to the place of purchase for disposal. The place of purchase will then recycle the device.

Rechargeable batteries/batteries:

Do not throw your rechargeable batteries/batteries in the domestic waste, in fire, or in water. Rechargeable batteries/batteries should be collected, recycled, or disposed of in an environmentally friendly manner.

Only for EU countries:

According to the directive 91/157/EEC, defect or used rechargeable batteries/batteries must be recycled. Waste rechargeable batteries/batteries can be directly handed to:

Promed GmbH, Lindenweg 11, D-82490 Farchant.

This product was produced with the utmost care and was thoroughly tested before it left the factory. Therefore, we provide a warranty period of 24 months from the date of purchase for this product. **Promed** products conform to their description and the relevant specifications; it is your responsibility to ensure that the product which you have purchased is designed for your intended use.

- In the event of material defects and/or manufacturing errors which appear during proper use and are detected during the warranty period, we will replace all defective parts of the product free of charge within the warranty period, including the labour costs for warranty repairs.
- **The warranty does not cover:**
 - Normal wear on the product
 - Defects which were caused during transportation or storage of the product
 - Defects or damage which were caused through improper use or inadequate maintenance
 - Damage resulting from disregarding the notes in the operating manual
 - Damage resulting from modifications to the product which were not carried out by **Promed**.
 - Damage caused by sharp objects or as a result of torsion, compression, being dropped, an abnormal impact or other actions which are beyond **Promed's** reasonable control.
 - Parts which are subject to wear (e.g. moving parts such as ball bearings, etc., fasteners, etc.) are generally excluded from the warranty.
- **A claim under the warranty is not enforceable if:**
 - the product is not sent back in its original packaging or in adequately secure packaging;
 - it has been modified or repaired by a person or company other

than **Promed** or one of **Promed's** authorised dealers;

- the product has been repaired using parts which are not approved by **Promed**;
- the serial number / LOT number of the product has been removed, deleted, changed or made illegible.
- For reasons of hygiene, any products which have been exposed to direct contact with a body or with bodily fluids (e.g. blood) must be packaged in an additional plastic bag before they are sent back. In this case, a specific note of this situation must be made in the package or in the accompanying documents.
- For components or products which are replaced during the warranty period, a warranty is only provided for the remainder of the original warranty period; provided that the product or part replacement was carried out by **Promed** or one of **Promed's** authorised dealers.
- The warranty period begins on the day of purchase. Claims under the warranty must be established within the warranty period. Complaints arising after the expiry of the warranty period cannot be considered.
 - Under these warranty conditions, the warranty shall only apply if the date of purchase can be proven by means of a sales receipt or similar.
 - We reserve the right to make technical and visual changes as well as changes to the features.
- This warranty is only valid and enforceable in the country in which it was purchased by the original buyer, provided that it was **Promed's** intention that the product should be offered for sale in this country. This warranty is also enforceable in any country within the European Economic Area in which **Promed** has an authorised importer or sales partner. Depending on each country, specific and differing warranties and guarantees may

be applicable owing to the particular applicable legislation. These legal provisions are neither excluded nor limited by these warranty conditions. To the extent permissible under national law, the warranty period shall not be extended, renewed or otherwise affected by any subsequent resale, repair or replacement of the product.

- The provisions of the UN Convention on Contracts for the International Sale of Goods shall not apply.
- Our warranty conditions do not affect the seller's legal warranty obligation.
- To the fullest extent possible under the applicable mandatory legislation, these warranty conditions constitute your sole and exclusive legal remedy and shall apply in place of any other express or implied warranty conditions. **Promed** is not liable for any unusual, incidental, punitive or consequential damages including, but not limited to loss of profits, loss of use, loss of revenue, costs of replacement equipment or facilities, third party insurance claims, and damage to property which are the result of the purchase or use of the product or occur owing to a breach of warranty, a breach of contract, negligence, product defects or any other legal or statutory circumstances, even though **Promed** was aware of the possibility of such damages. **Promed** is not liable for any delay in warranty claims.
- **Promed** cannot be held liable for any possible translation errors.
- The following information is vital for problem-free processing:
 1. Original sales slip/receipt or dealer's stamp with the date of purchase
 2. Defect detected
 3. Unit name / type

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Well-Life Healthcare Ltd.
MDSS GmbH, Schiffgraben 41, D-30175 Hannover, Germany
Promed GmbH · Lindenweg 11 · D-82490 Farchant